

PERFORMING A RESCUE

1



Is patient responsive?

2



Call emergency services!

3



Get AED.

4



Place AED on ground.
Open lid.

5



Remove clothing.
Expose chest.

6



Attach pads.

7



Do not touch the patient.

8



If required, AED will deliver
shock automatically.

9



If present, place CPR device.

10



Follow AED prompts.
Give compressions.

11



If instructed, give breaths.

Automatic AED



Cardiac Science and the Shielded Heart logo are trademarks of ZOLL Medical Corporation.
Copyright © 2021 All Rights Reserved.
www.zoll.com

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA USA
01824-4105

EC REP

ZOLL International Holding B.V.
Rooftopweg 18
6592 PV ELST
The Netherlands



CE
0123



70-02105-01 A

