

# Instructions for Use Hi-Luck 775 N99 Particulate Respirator


Before use of this respirator, the wearer must read and understand these user instructions.

## Fitting Instructions

Before use for respiratory protection, a written respiratory protection program must be implemented meeting all the requirements of OSHA 29 CFR 1910.134.

Notch

1. Hold the respirator in your hand with the narrow end at your fingertips, allowing the headbands to hang freely below your hand. Position the respirator on your face with the notched facepiece lip over the bridge of your nose.

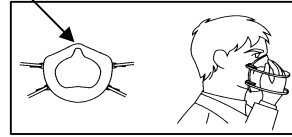

2. Pull the top strap over your head, positioning it high at the top back of your head. Pull the bottom strap over your head and position it around the neck below the ears.



3. Adjust the respirator position by shaking gently the respirator from side to side or up and down. If necessary, adjust the headband tension. To tighten the headband, take both the ends of each strap between the fingers of the respective hands. To loose the headbands, remove the respirator and pull each strap backward through slits of the buckles.



4. Perform a User Seal check prior to each wearing. To check fit, place both your hands over the respirator and exhale. If air leaks around the nose, readjust the respirator position on your face. Be careful not to disturb the respirator position. If air leaks at the respirator edges, remove the respirator, adjust the headband tension and repeat the procedures described in the step 1 through 4.

## Time of Replacement or Change of Respirator

If the respirator becomes damaged, distorted, soiled or notably wet or breathing becomes difficult, leave the contaminated area and dispose of the respirator in a safe manner consistent with good industrial hygiene practices and applicable regulations.

## Storage

Store the respirator in a cool, dark, dry area to avoid direct sunlight, contaminant, excessive moisture, heat, cold and physical damage.

## Warning

1. Do not use for protection against oil aerosols, gases or vapors, asbestos, paint spray mist or sandblasting.
2. Follow all user instructions on the use of this respirator. Also, wear this respirator during all the times of exposure. If not, it reduces respirator effectiveness and may cause sickness or death.
3. Leave the contaminated area immediately and consult a licensed medical physician if breathing becomes difficult or if dizziness, nausea, irritation or other distress occurs.
4. Do not use in unventilated or inadequately ventilated area containing less oxygen less than 19.5% oxygen. This respirator does not supply oxygen.
5. Do not use where concentrations of contaminants are unknown or immediately dangerous to life and health or where concentrations exceed the maximum exposure limits prescribed in applicable government standards or regulations, whichever is lower.
6. Do not wear if respirator does not fit properly. Refer to Fitting Instructions 4.

**NIOSH approval label of this respirator can be found on the container.**

**Expiration date for use is 5 years from the date of manufacture. The date of manufacture is printed on the mask.**