FILTER TECHNOLOGY

ENGLISH

INSTRUCTIONS FOR USE

AIR-PURIFYING FILTERING FACEPIECE RESPIRATOR

» The Segre Safety Respirator is approved as a filtering facepiece in accordance with the NIOSH regulation 42 CFR Part 84.

APPLICATIONS

» The Segre Safety Respirator gives protection against particulate aerosols free of oils at N95 or N99 filter efficiency level. Refer to markings on the individual respirator for the level of protection.

Table with 3 columns: Filter Type, Minimum Efficiency, Applications. Rows for N99, N95, and N95.

» Guidelines for the selection and use may vary between countries, always check prior to use to ensure that your selection for the intended application meets national requirements.

FITTING INSTRUCTIONS

① Remove the respirator from the sealed bag. Check that the lengths of the upper and lower headband straps are equal. ② The length of the strap may need shortening to create a good facial seal. This can be done by winding them around the side clip. Recommended adjustment: Small head = 2 wraps, Medium head = 1 wrap and Large head = 0 wraps per top and bottom strap. ③ Open the respirator to 90° then hold the respirator in your hand with rubber straps hanging forwards over the front of the nose clip. Place the respirator over your nose and mouth. ④ Place the lower rubber strap down to your neck. Place the upper rubber strap to the crown of your head. ⑤ Using the fingers of both hands, carefully shape the nose clip to fit your nose and ensure a good seal is created and adjust the respirator to achieve completely tight fit to your face.

» Test the tightness of the facial fit: 1. Using your finger and thumb in the centre of the respirator pinch with a firm grip and carefully pull away from your face. If it is easy to do this then you should increase the tension of the strap as advised in point 2. 2. Blow air through the respirator. If air leaks out around the nasal area (nose clip) or elsewhere, these leaks must be eliminated by readjusting the nose clip, strap tension and respirator position, before entering the contaminated area.

NOTE: OSHA regulations require that all users be fit tested prior to use either qualitatively or quantitatively.

CLEANING & MAINTENANCE

» Replace the respirator if it is damaged or if the breathing become difficult. WARNING: Do not re-use the respirator. Do not wash the respirator. Cleaning may degrade some respirator components and reduce respirator effectiveness. Inspect all respirator components before each use to ensure proper operating condition. Failure to do so may result in death.

STORAGE

» The respirator should be stored and transported out of direct sunlight, away from sources of high temperature, in a cool dry atmosphere at temperature between -20°C to 40°C, in an uncontaminated environment and in the original packaging.

» Products must be discarded when they reach their shelf life date. Storage or transportation under conditions other than those specified may affect shelf life. Dispose of the respirator according to the filtered substance in accordance with current waste treatment regulations.

CAUTIONS & LIMITATIONS

- A - Not for use in atmospheres containing less than 19.5% oxygen. B - Not for use in atmospheres immediately dangerous to life or health. C - Do not exceed maximum use concentrations established by regulatory standards. J - Failure to properly use and maintain this product could result in injury or death. M - All approved respirators shall be selected, fitted, used and maintained in accordance with MHSA, OSHA and other applicable regulations. N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer. O - Refer to users instructions and/or maintenance manuals for information on use and maintenance of these respirators. P - NIOSH does not evaluate respirators for use as surgical masks. Δ Before occupational use of this respirator a written respiratory protection program must be implemented meeting all the local government requirements. In the United States employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.

Information icons: See information supplied by manufacturer, Humidity range of storage conditions (<80% RH), Temperature range of storage conditions (-20°C to +40°C), Expiry Date.

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FRANÇAIS

ESPAÑOL

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

GVS Filter Technology UK
Vickers Industrial Estate
Morecambe, Lancashire, UK
Tel: +44 (0) 1524 847600

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

RESPIRATOR COMPONENTS					
TC -	PROTECTION ¹	FACEPIECE			CAUTIONS AND LIMITATIONS ²
		N30000	N31000	N21000	
84A-9110	N99	X			ABCJMNOP
84A-9111	N99		X		ABCJMNOP
84A-9449	N95			X	ABCJMNOP

1. PROTECTION

N99 - Particulate Filter (99% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.
N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

2. CAUTIONS AND LIMITATIONS

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to users instructions and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.

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