FITTING INSTRUCTIONS

- Hold mask in one hand with straps around back of hand. Place over mouth and nose with nose clip over bridge of nose. Pull lower strap over head and fit around neck. Pull upper strap over head and place it above ears.
- Form nose clip around nose and adjust tension in straps by positioning straps on head until you obtain a positive face seal.
- Perform a User Seal Check prior to each wearing. It is the wearer's responsibility to obtain a proper fit. Each time respirator is used, wearer must test fit by cupping both hands over the respirator and exhaling vigorously. If air is felt flowing around nose or leaking around edges of the respirator, tighten the nosepiece and reposition or tighten straps until air is no longer escaping when wearer exhales. If you CANNOT achieve a proper face seal, DO NOT enter the contaminated area. See your supervisor.

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LIMITED WARRANTY – Manufacturer expressly limits its warranty for this product to the physical standard and specifications set forth on this package. This limited warranty is in lieu of all other warranties, expressed or implied, including, but not limited to merchantability and fitness for particular purposes. It is the responsibility of user to determine the suitability of the product for its intended use based on the physical standard and specifications set forth hereon, and to assume all the risks and liability in connection therewith.

Made in China

Gateway Safety, Inc.
11111 Memphis Avenue, Cleveland, Ohio 44144
800-822-5347

These respirators are approved only in the following configurations:

TC-	Protection ¹	Respirators	Cautions and Limitations ²
84A-3907	N99	X	ABCJMNOPS

1. Protection

N99 Particulate filter (99% filter efficiency level)
effective against particulate aerosols free of oil;
time use restrictions may apply.

2. Cautions and Limitations

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to User Instructions and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.
- S - Special or critical User Instructions and/or specific use limitations apply. Refer to User Instructions before donning.

USE RESTRICTIONS

- DO NOT USE in spray painting or sandblasting operations.
- DO NOT USE for protection in asbestos environments.
- DO NOT USE for protection against vapors or gases.
- DO NOT USE for welding applications or near any welding activities.
- DO NOT USE against any contaminant whose concentration is unknown, or immediately dangerous to life.
- DO NOT USE against concentrations of Dust and Mist which exceed the maximum use concentration or 10 times contaminant's permissible exposure limit established by OSHA.
- USE AGAINST non-oil particulate aerosols, such as coal, iron ore, cotton, flour, and for sprays that do not emit vapors or gases.
- S - Special or Critical User Instructions - This Gateway Safety filtering face piece respirator Model #80402V is marketed and sold by Gateway Safety, Inc. under a production agreement with Jinlyu Industrial Co., Ltd. in accordance with NIOSH TC-84A-3907.

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TruAir®
N99 Vented Respirator
NIOSH APPROVED

WARNING

- A. Before occupational use of this respirator, a written respiratory protection program meeting all the local government requirements must be implemented. In the United States employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.
- B. This respirator does not supply oxygen. Use only in well ventilated areas containing sufficient oxygen, at least 19.5%, to support life.
- C. Do not use when concentrations of contaminants are unknown; at levels dangerous to health and life, in excess of 10 times the permissible exposure limit (PEL) or in excess of other regulations.
- D. Leave contaminated area immediately if respirator is damaged, or if any physical distress such as breathing difficulty, dizziness or nausea occurs.
- E. Do not alter, modify or misuse respirator.
- F. Facial hair, beards or certain facial characteristics may prevent a proper fit and thus prevent the use of this respirator.
- G. Keep unused respirators in the display box, stored in a cool, dry, uncontaminated area, away from direct sunlight until use.
- H. Dispose of respirator after single use, increased breathing resistance or hygiene damage.

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N99
NIOSH APPROVED
42CFR84 Approved Product

Box of 10

Gateway Safety, Inc.
11111 Memphis Ave.
Cleveland, OH 44144
800-822-5347
GatewaySafety.com

Part #80402V

12.75x21.9x12.75