## BIELCOR B300 (N99)

## **Donning Procedures**

- 1. Hold the respirator in one hand with the nosepiece at your fingertips, allowing the headbands to hang below your hand.
- 2. Press the respirator against your face with the nosepiece on the bridge of your nose.
- 3. Place the top band high on the back of your head. Move the bottom band over your head and position it below your ears.
- 4. Using both hands, mould the nosepiece to the shape of your nose.
- 5. Test the fit. Cup both hands over the respirator and inhale vigorously. If no negative pressure is felt inside the respirator, readjust the position of the respirator according to steps #2, #3, and #4 till get a better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.

