175X130X190mm

10 **66N HSOIN**

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KleenGuard
BRAND

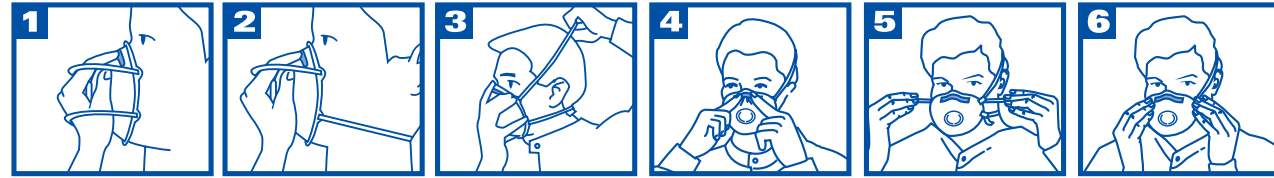
64520

M30 Particulate Respirator with Valve N99
NIOSH N99

(EN) Effective against oil-free particulate aerosols and dust. Before occupational use, OSHA requires the user be fit tested with this model of respirator per regulation 29 CFR 1910.134.

IMPORTANT: Follow all instructions and warnings on the use of this respirator. Wear during all times of exposure. Failure to do so will reduce respirator effectiveness and user protection, and may result in sickness or death.

Time use limitations: If respirator becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and properly dispose of the respirator.

INSTRUCTIONS FOR USE:

Follow these instructions each time the respirator is worn. (See pictures 1-6)

1. Hold the respirator in your hand with head straps hanging forwards over the front of the nose clip. Place the respirator over your nose and mouth.
2. Place the bottom elastic strap over your head just below the ears. Untwist the strap.
3. Place the top head strap over your head above your ears. Untwist the strap.
4. Using both hands, carefully shape the nose clip to fit your nose and adjust the respirator to achieve a completely tight fit to your face.
5. Strap tension may be reduced without removing the respirator from the head by pushing out on the back of the buckle. To tighten, pull the strap end through the buckle.
6. Test the tightness of the facial fit by covering the front of the respirator with both hands and inhale sharply. If air leaks out around the nasal area (nose clip) or elsewhere, these leaks must be eliminated by readjusting the nose clip and respirator, before entering the contaminated area.

Kimberly-Clark
 1400 Holcomb Bridge Road
 Roswell, GA 30076
www.kcprofessional.com

This respirator is approved only in the following configuration:

TC-84A-4554	Protection ¹ N99	Respirator 64520	Cautions and Limitations ² ABCJMNOPS

1) **Protection:** N99-Particulate Filter (99% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

2) **Cautions and Limitations:**

- A. Not for use in atmospheres containing less than 19.5 percent oxygen.
- B. Do not use in atmospheres immediately dangerous to life or health.
- C. Do not exceed maximum use concentrations established by regulatory standards.
- J. Failure to properly use and maintain this product could result in injury or death.
- N. All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- O. Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- P. Refer to user's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- F. NIOSH does not evaluate respirators for use as surgical masks.
- S. Special or critical User's Instructions and/or specific use limitations apply. Refer to User's Instructions before donning.

WARNING!

1. Do not use for sandblasting or to protect against asbestos, lead-based paint, pesticides, oil aerosols, vapors or gases.
2. Do not wear this respirator when conditions prevent a good facial seal. It is important to fit check the respirator each time it is worn.
3. This respirator is not for medical use.
4. Never alter or modify this product.
5. Special Instructions: The model 64520 filtering facepiece respirator has been manufactured by Makrite Industries for Kimberly-Clark under TC-84A-4554.

KleenGuard
BRAND

64520

M30 Particulate Respirator with Valve N99
NIOSH N99

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© 2007 KCIWV.
® / * Trademarks of Kimberly-Clark Worldwide, Inc.