moldex Page 1 of 1

Toll free number (800) 421-0668



Ideas That Wear Well.®

Home / Handy Strap



HandyStrap[®] Fitting Instructions



1

neck with shell against

chest.

2



3



Attach buckle behind

a) Fit mask to face and pull top of strap to crown of head. Nose cushion must not be creased inside of

respirator.

b) Always use top and bottom part of

strap.
Adjust tension for a comfortable fit. When changing from any model/size to another respirator, you must fit test.

te and Each time user enters contaminated area respirator must be seal be checked.
Cover front of respirator by cupping both hands.

INHALE SHARPLY.
A negative pressure should be felt inside respirator.
If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator.

sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.

To hang around neck: a) Undo buckle.

4

- b) Allow strap to fall around neck.
- c) Pull mask down to chest.

OR

b) Pull top strap forward away from head.c) Adjust tension to

c) Adjust tension to allow mask to hang freely.

CAUTION: If used against biohazards, discard immediately after removing from face and do not hang around neck.

TOP