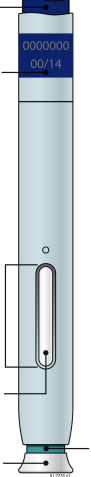
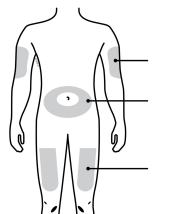
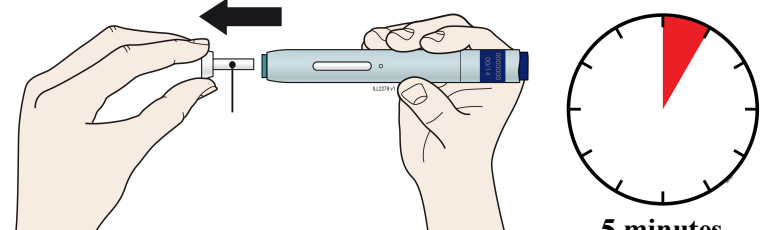


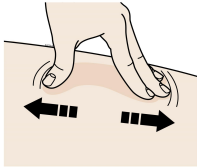
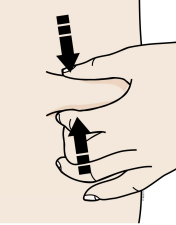
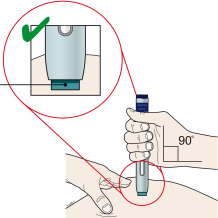
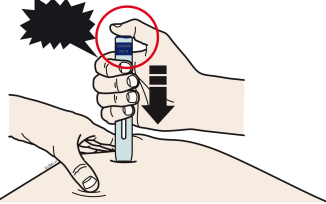
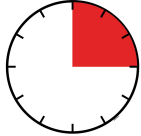
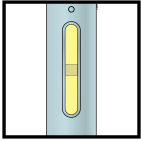
**REFERENCE GUIDE**

<b>Side 1</b>	<b>Reference Guide</b>
	Aimovig® (AIM-oh-vig) (erenumab-aooe) injection, for subcutaneous use Single-Dose Prefilled SureClick® Autoinjector 70 mg/mL

**Read all instructions in carton before use.**

**Questions? Call 1-800-77-AMGEN  
(1-800-772-6436)**

<b>Guide to parts</b>			<b>Wait at least 30 minutes</b> for the autoinjector to come to room temperature before giving your injection.	<b>2</b>	Pull the white cap straight off only when you are ready to inject. <b>Do not</b> leave the white cap off for <b>more than 5 minutes</b> . This can dry out the medicine.
<div style="display: flex; justify-content: space-between;"> <div style="width: 45%;"> <p>Purple start button</p> <p>Expiration date</p> <p>Window</p> <p>Medicine</p> <p>White cap on</p> </div> <div style="width: 45%; text-align: center;">  <p>Green safety guard (needle inside)</p> </div> </div>	<b>1</b> Prepare and clean your injection site.		 <div style="display: flex; justify-content: space-around; margin-top: 10px;"> <div style="text-align: center;">Upper arm</div> <div style="text-align: center;">Stomach</div> <div style="text-align: center;">Thigh</div> </div>	 <p style="text-align: center;">White cap</p> <p style="text-align: center;"><b>5 minutes</b></p>	
<b>Turn over to continue</b>					

<p><b>Side 2</b></p>	<p><b>Reference Guide</b>  Aimovig® (AIM-oh-vig)  (erenumab-aooe)  injection, for subcutaneous use  Single-Dose Prefilled SureClick® Autoinjector  70 mg/mL</p>	<p><b>AMGEN</b>  Manufactured by:  Amgen, Inc.  Thousand Oaks, CA 91320  © 2020-2022, 2024 Amgen Inc. All rights reserved.  ♻️ This printed material is recyclable.  &lt;part number&gt; Revised: 8/2024 v4</p>	<p><b>Read other side first</b></p>	
<p>Stretch <b>or</b> pinch your injection site to create a firm surface.</p>		<p><b>Put</b> the green safety guard (needle inside) on your skin at 90 degrees.</p>	<p>Firmly <b>push</b> down the autoinjector onto your skin until it stops moving.</p>	<p>Keep <b>pushing</b> down on your skin for <b>about 15 seconds</b>.</p>
<p><b>3</b></p>	<p><b>Stretch method:</b></p>  <p><b>or</b></p> <p><b>Pinch method:</b></p> 	<p><b>4</b></p> <p>Green safety guard (needle inside)</p> 	<p><b>5</b></p> <p>When you are ready to inject, <b>press</b> the purple start button. You will hear a click.</p> <p>"click"</p> 	<p><b>6</b></p>   <p><b>15 seconds</b></p> <p>Window turns from clear to yellow when the injection is done.</p> <p>"click"</p> 